
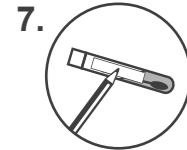
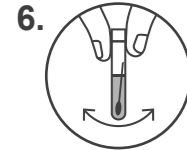
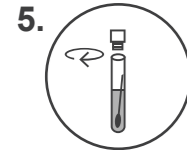
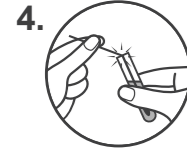
SalivaGene Swab Comfort

REF 1035231100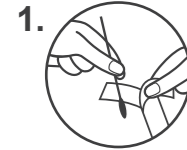

DE 808.02
2025-06-18

DEUTSCH

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info@invitek.com
www.invitek.com

ENGLISH

SalivaGene Swab Comfort Instructions

For In Vitro Diagnostic Use*

*) In compliance with REGULATION (EU) 2017/746 on in vitro medical devices (Not for in vitro diagnostic use in countries where the REGULATION (EU) 2017/746 on in vitro medical devices is not recognized.)

INTENDED USE:

Collection of human oral samples for DNA stabilization and extraction (using the SalivaGene DNA Extraction Kits).

The DNA of swab samples collected using SalivaGene Swab Comfort is stable for at least 6 months at room temperature in the stabilization buffer. Long-term storage of the sample (> 6 months) should be done at -20 °C.

Children under the age of 14 and people with disabilities who cannot independently understand/execute the use of the product must be instructed accordingly.

CONTENTS:

Swab, Stabilizer tube, label for donor description. The Stabilizer tube contains approx. 650 µl of liquid DNA Stabilizer reagent.

LIMITATIONS OF THE PROCEDURE:

Do not eat, drink, smoke or chew gum at least 30 min prior to sample collection. Ensure the swab tip is not exposed to any surface prior to collection.

Put the Stabilizer tube upright to prevent the liquid inside the tube from spilling. Be sure to move the swab over the entire cheek and to moisten it with saliva.

1. Open swab package and remove swab without touching the tip.

2. Insert swab into the mouth and rub swab tip firmly against the inner cheeks for about 30 seconds on each side.

3. Unscrew lid from Stabilizer tube and put it aside for further use. Insert swab into the Stabilizer tube.

4. Break swab at breaking point. Discard the broken handle part.

5. Screw lid tightly onto the Stabilizer tube again.

6. Gently shake tube for 15 sec to mix buccal cells and DNA Stabilizer reagent.

7. Label Stabilizer tube with donor name and collection date.

SHIPMENT:

Ship according to locally applicable guidelines and regulations such as UN 3373 (Biological Substance, Category B).

SAFETY PRECAUTIONS:

Keep out of the reach of children. Small cap: choking hazard. Caution should be used when inserting swab into the mouth. Do not use the SalivaGene Swab Comfort if content of the tube is spilled.

Do not drink the DNA Stabilizer reagent. Wash with water if DNA Stabilizer reagent comes in contact with skin or eyes.

WARNING: Hazard statements

H319: Causes serious eye irritation.
H412: Harmful to aquatic life with long lasting effects.

Precautionary statements

P264: Wash hands, forearms and face thoroughly after handling.

P273: Avoid release to the environment.

P280: Wear protective gloves/protective clothing/eye protection/face protection/hearing protection.

P305+P351+P338: IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.

P337+P313: If eye irritation persists: Get medical advice/attention.

P501: Dispose of contents/container to hazardous or special waste collection point, in accordance with local, regional, national and/or international regulation.

Any serious incident must be reported to the competent authority.

LABEL LEGEND:

Lot number	Unique identifier of a medical device
Catalogue number	Expiry date
Consult operating instructions	Store at room temperature 15 – 30 °C
Do not reuse	Amount of sample collectors
In vitro diagnostic medical device	CE Marking of Conformity

Manufacturer
ALS Life Sciences Portugal, S.A.
Zona Industrial de Tondela, ZIM II, Lote 6,
3460-070 Tondela, Portugal

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